

## Ureteral Access Sheath Precaution and Warning

- 1) The device is sterilized by ethylene oxide, it's for single use only, and prohibited for re-sterilized.
- 2) Do not use the device if the package is damaged, contaminated or wet.
- 3) The device should only be used by physicians who have comprehensive knowledge of the indications, techniques and risks of the procedure.
- 4) Do not use the device after its expiry date.
- 5) This product with negative pressure suction device maximum negative pressure should be controlled within -10kPa.
- 6) Do not advance access sheath without dilator in place.
- 7) If resistance is encountered, suspend progression of the access sheath or guidewire until the cause has been determined and corrective actions have been performed.
- 8) In the event of a tight stenosis, consider ureteral balloon dilatation prior to insertion of the access sheath.
- 9) The device should be disposed as medical waste after use.